



Pregnancy Tests



How it works

The pregnancy tests detect elevated levels of Human Chorionic Gonadotropin (hCH) in the urine which occurs when an egg is fertilised. Tests can be carried out from the 1st day of a missed period. Detailed instructions are included to enable the testing to be carried out at the appropriate time.

Tests are carried out by inserting the tip of the test into urine flow (midstream) or into a sample of urine (strip) and waiting for 1-5 minutes to check the number of lines in the test window as shown in the instructions.

SURESIGN tests are 10mIU/ml which allows testing to be carried out 4-5 days earlier than QUIK-CHECK, at approximately 10 days after conception.

Contents

Instruction Sheet		1
Tests per Pack	QUIK-CHECK *	1
	SURESIGN	2
Effectiveness		>99%

Packing Details

Product code	Description	Tests per Retail Pack	Retail Packs per Inner	Retail Packs per Shipper
FHC 101-1	QUIK-CHECK Pregnancy Strip Single	1	12	540
SFHC 101-2	SURESIGN Pregnancy Strip Twin	2	6	288
FHC 103-1	QUIK-CHECK Pregnancy Midstream Single	1	8	224
SFHC 103-2	SURESIGN Pregnancy Midstream Single	2	6	180

* **Quik-Check is available in the UK and Ireland only**



Sales - UK
 The Miles Group Ltd.
 Tel 01484 536344
 Fax 01484 536220
 info@milesgroup.co.uk
 www.milesgroup.co.uk

Sales - Worldwide
 Ciga Healthcare Ltd
 Tel 02825 685385
 Fax 02825 685145
 info@cigahealthcare.com
 www.cigahealthcare.com



Diabetes Screening



How it works

Diabetes Screening tests detect the presence of Glucose in the urine which is an indicator of diabetes. Since the onset of diabetes can occur at any time it is essential that tests are carried out regularly.

Contents

Instruction Sheet		1
Colour Chart		1
Tests per Pack	SURESIGN	3
Effectiveness		>99%

Packing Details

Product code	Description	Tests per Retail Pack	Retail Packs per Inner	Retail Packs per Shipper
SDIA 101	SURESIGN Diabetes Screening	1	6	288



Sales - UK
The Miles Group Ltd.
Tel 01484 536344
Fax 01484 536220
info@milesgroup.co.uk
www.milesgroup.co.uk

Sales - Worldwide
Ciga Healthcare Ltd
Tel 02825 685385
Fax 02825 685145
info@cigahealthcare.com
www.cigahealthcare.com



Ovulation Tests



How it works

The Easy Prediction One Step Ovulation Test is a rapid, one-step lateral flow immunoassay for the qualitative detection of Lutenising Hormone surges in urine, signalling that ovulation is likely to occur in the next 24-36 hours. The test utilizes a combination of antibodies including a monoclonal LH antibody to selectively detect elevated levels of LH. The assay is conducted by urinating on the Absorbent Tip or immersing the Absorbent Tip in urine, and obtaining the result from the coloured lines.

Contents

Instruction Sheet		1
Tests per Pack	QUIK-CHECK*	5
	SURESIGN	5
Effectiveness		>99%

Packing Details

Product code	Description	Tests per Retail Pack	Retail Packs per Inner	Retail Packs per Shipper
FLH 103-5	QUIK-CHECK Midstream Ovulation	5	4	96
SFHC 103-5	SURESIGN Midstream Ovulation	5	3	90

* *Quik-Check is available in the UK and Ireland only*



Sales - UK
The Miles Group Ltd.
Tel 01484 536344
Fax 01484 536220
info@milesgroup.co.uk
www.milesgroup.co.uk

Sales - Worldwide
Ciga Healthcare Ltd
Tel 02825 685385
Fax 02825 685145
info@cigahealthcare.com
www.cigahealthcare.com



Thermometer



How it works

The INFRARED thermometer measures body temperature in the ear or on the forehead.

- Easy to use
- Results in 1 second
- Easy to read large display
- Suitable for babies and children

The DIGITAL thermometer measures body temperature in Oral, Underarm or Rectal use.

- Easy to use
- Easy to read large display
- Beeps when reaches peak temperature
- Automatic recall of last measurement reading
- Waterproof tip

Contents

Instruction Sheet		1
Stand/Cover	Infrared/Digital	1
Spare Battery	Infrared	1
Units per Pack	Digital	1
	Infrared	1
Accuracy		+/-0.1 °C

Packing Details

Product code	Description	Units per Retail Pack	Retail Packs per Inner	Retail Packs per Shipper
OT 11-111	SURESIGN Digital	1	6	288
SINFR 107	SURESIGN Infrared	1	1	144



Sales - UK
 The Miles Group Ltd.
 Tel 01484 536344
 Fax 01484 536220
 info@milesgroup.co.uk
 www.milesgroup.co.uk

Sales - Worldwide
 Ciga Healthcare Ltd.
 Tel 02825 685385
 Fax 02825 685145
 info@cigahealthcare.com
 www.cigahealthcare.com



Menopause Tests



How it works

The Easy Detection One Step Menopause Test is a rapid, one-step lateral flow immunoassay for the qualitative detection of FSH in urine to aid in the detection of menopause. The test utilizes a combination of antibodies including a monoclonal FSH antibody to selectively detect elevated levels of FSH. The assay is conducted by urinating on the Absorbent Tip or immersing the Absorbent Tip in urine, and obtaining the result from the coloured lines.

Contents

Instruction Sheet		1
Tests per Pack	QUIK-CHECK*	2
	SURESIGN	2

Effectiveness >99%

Packing Details

Product code	Description	Tests per Retail Pack	Retail Packs per Inner	Retail Packs per Shipper
FFS 103	QUIK-CHECK Midstream Menopause	2	6	240
SFSH 103	SURESIGN Midstream Menopause	2	6	180

* **Quik-Check is available in the UK and Ireland only**



Sales - UK
The Miles Group Ltd.
Tel 01484 536344
Fax 01484 536220
info@milesgroup.co.uk
www.milesgroup.co.uk

Sales - Worldwide
Ciga Healthcare Ltd
Tel 02825 685385
Fax 02825 685145
info@cigahealthcare.com
www.cigahealthcare.com



Cholesterol Tests



How it works

The Cholesterol Home Screening Tests includes test strips that are treated with chemical reagents. A whole blood sample is absorbed on a receiving pad on the back of a test strip. The blood sample absorbs through to the front of a test strip and reacts with the reagents. The chemistry used is the very well established cholesterol esterase/cholesterol oxidase enzymatic reaction to produce hydrogen peroxide followed by the peroxidase catalyzed reaction hydrogen peroxide with a colourless indicator which thereby produces colour. The colour produced on the reaction pad can be related to the cholesterol concentration in the blood by comparison with a supplied colour chart. The deeper the colour is, the higher the Cholesterol level.

The Total test is for total cholesterol in the blood the Total++ test is for both total and HDL and by calculation LDL.

Contents

Instruction Sheet	1
Prep Pad	1
Lancet	1
Plaster	1
Colour Chart	1
Tests per Pack	SURESIGN 1

Packing Details

Product code	Description	Tests per Retail Pack	Retail Packs per Inner	Retail Packs per Shipper
SCHO 105	SURESIGN Cholesterol Total ++	1	6	288
SCHO 104	SURESIGN Cholesterol Total	1	6	288




Sales - UK
 The Miles Group Ltd.
 Tel 01484 536344
 Fax 01484 536220
 info@milesgroup.co.uk
 www.milesgroup.co.uk

Sales - Worldwide
 Ciga Healthcare Ltd
 Tel 02825 685385
 Fax 02825 685145
 info@cigahealthcare.com
 www.cigahealthcare.com



Blood Pressure Monitor

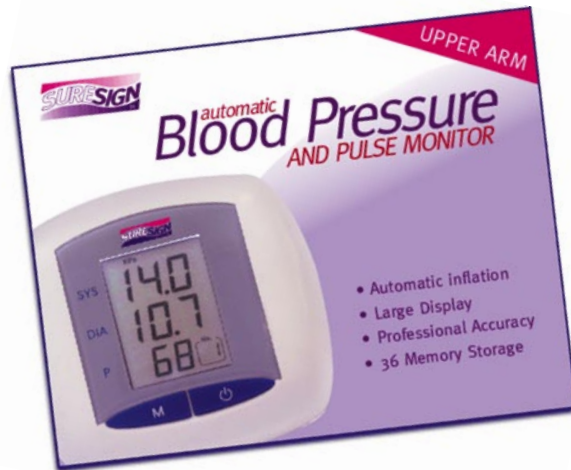

How it works

The blood pressure monitor is a professional upper arm monitor with a full cuff which also takes pulse measurements. It is fully automatic with an internal pump for inflation and deflation and requires no manual intervention to take measurements and display the readings, which can be in European or American formats. It is very accurate and reliable and is easy to use and can store 36 previous readings.

Contents

Instruction Sheet	1
Monitor	1
Standard Cuff (Large cuff available as an accessory)	1

Packing Details

Product Code	Description	Units per Retail Pack	Retail packs per inner	Retail packs per shipper
BPMON 107	Blood Pressure and pulse Monitor	1	1	36



Sales - UK
The Miles Group Ltd.
Tel 01484 536344
Fax 01484 536220
info@milesgroup.co.uk
www.milesgroup.co.uk

Sales - Worldwide
Ciga Healthcare Ltd
Tel 02825 685385
Fax 02825 685145
info@cigahealthcare.com
www.cigahealthcare.com